

Hearing enhancement for those who **do not** use hearing aids

Neck loop and Headphones

1. Unplug and remove black loop and replace with black fabric strap
2. Insert battery
3. Turn on
4. Select channel 1 for show relay*, 2 for audio description or 1 and 2 for show relay and audio description. (Both lights on the top of the neck loop should be lit to indicate both channel 1 and 2 have been selected)
5. Plug in headphones
6. Turn wheel to change volume

* The show relay take any sounds from the stage (dialogues, sound effects etc.) and amplifies it for the neck loop/Stethoset user.





OR


Stethoset

1. Insert battery
2. Turn on
3. Select channel 1 for show relay*, 2 for audio description or the Infinity Symbol (∞) for Show relay & Audio Description
4. Turn wheel to change volume

* The show relay take any sounds from the stage (dialogues, sound effects etc.) and amplifies it for the neck loop/Stethoset user.

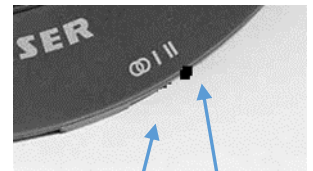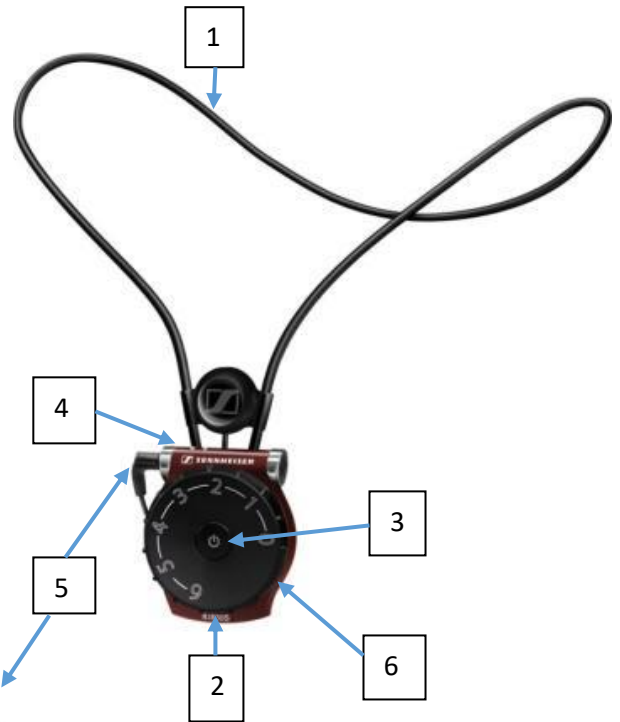

3

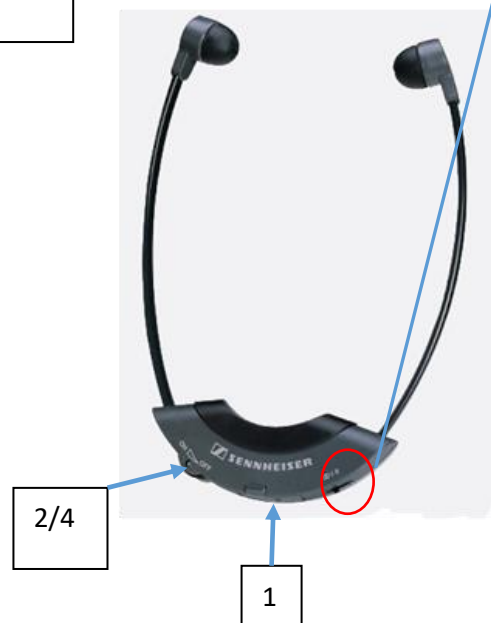

Hearing enhancement for those who use hearing aids.


Hearing aids **must** have “T” setting active

The “T” setting, this must be activated by an Audiologist

Neck Loop

1. Insert battery
2. Turn on
3. Select channel 1 for show relay*, 2 for audio description or 1 and 2 for show relay and audio description. (Both lights on the top of the neck loop should be lit to indicate both channel 1 and 2 have been selected)
4. Turn wheel to change volume

* The show relay take any sounds from the stage (dialogues, sound effects etc.) and amplifies it for the neck loop/Stethoset user.

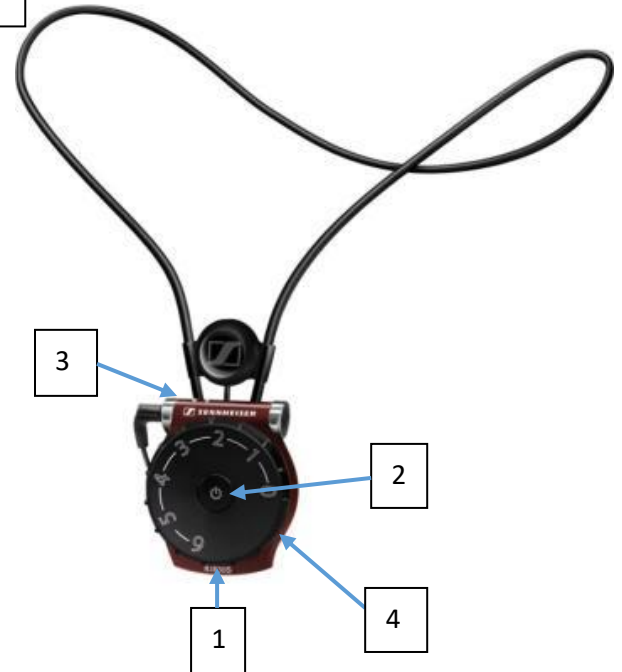


OR

1. Unplug and remove black loop and replace with black fabric strap
2. Insert battery
3. Turn on
4. Select channel 1 for show relay*, 2 for audio description or 1 and 2 for show relay and audio description. (Both lights on the top of the neck loop should be lit to indicate both channel 1 and 2 have been selected)
5. Plug in ear hooks
6. Turn wheel to change volume

* The show relay take any sounds from the stage (dialogues, sound effects etc.) and amplifies it for the neck loop/Stethoset user.

Neck Loop and Ear Hooks

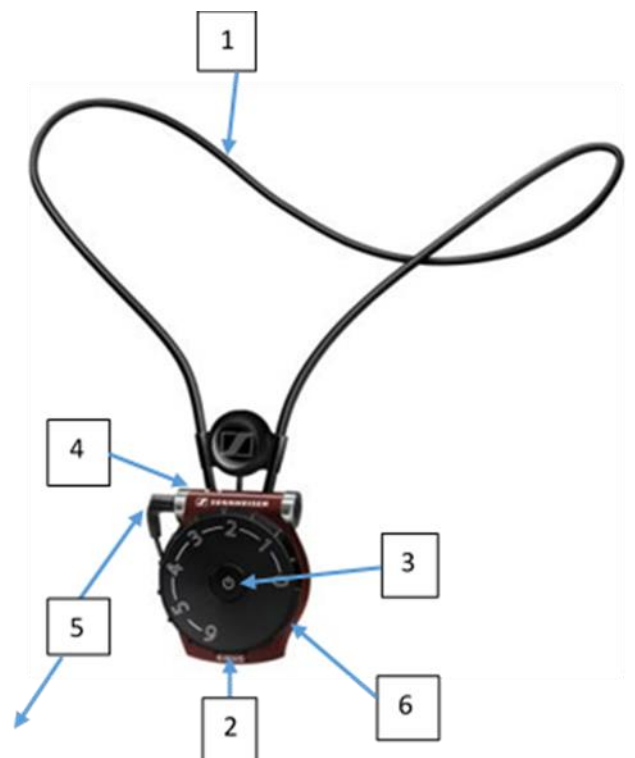

The ear hooks sit over the ear as shown in the pictures below. The hook will sit next to the hearing aid.



Remember:

- FOR NECK LOOPS ONLY. When you have selected a channel the amber light will flash when not in auditorium. When it connects inside this light will switch to a constant amber light and then go out. (It is still connected).
- Line of sight is very important. Make sure the Neck Loop or Stethoset is not covered or blocked by anything e.g. clothing, jewellery or hair. Signal can be disrupted when turning the volume wheel on the Neck Loop or if people are moving in front of the Neck Loop or Stethoset.
- **For both Audio Description and show relay, select channel 1 and 2 on the Neck Loop or the Infinity Symbol (∞) on the Stethoset.**
- **For shows that are not Audio Described, use Channel 1 for show relay.**
- The show relay take any sounds from the stage (dialogues, sound effects etc.) and amplifies it for the neck loop/Stethoset user.
- You can use your own headphones/earphones/ear hooks that have a standard 3.5mm jack. (The same the fits into mobile phones/laptops etc.)